

## Version 2:

- Ability to wake up the phone and answer a call using an external switch.
- Ability to phone contacts and receive calls.
- Ability to compose and send SMS text messages.
- Ability to receive and view SMS text messages.
- Ability to speak out text messages using Android Text-to-speech.
- Word prediction and Word anticipation when composing text.
- Ability to add new contacts.
- Ability to dial a number and enter numbers during a call.
- Ability to play music files on the phone.
- Ability to view Photographs.
- Ability to control most aspects of 3rd party Apps in expert mode.
- Ability to record and transmit InfraRed codes using HouseMate hardware.
- Ability to create, populate edit HouseMate grids with symbols and text.
- Ability to use finger movements accross the optic instead of using an external switch.
- Ability to tap anywhere on the ClickToPhone screen or softkeyboard instead of using an external switch.

## Version 3 Features:

- Ability to backup and restore user settings to SD card.
- Auditory scanning throughout the application
- Ability to select a Quick Text when sending SMS
- Ability to add and remove phrases from the Quick Text list
- Ability to launch Camera application from Homepage
- Ability to specify which application to launch for the Music Player, Gallery and Camera homepage links
- Ability to manage a Favorite list of applications
- Ability to reorder the cells of HouseMate grids via the touch screen
- Ability to password protect user settings

## Version 4 Features:

- Ability to speak the Quick Texts using the Text-to-speech engine
- New functions added to the Expert softkeyboard for controlling other applications
- Ability to launch ClickToPhone from the main Application list via a shortcut (can also be placed on homepage)

## Version 5 Features:

- Performance improved when launching other applications or browsing Favorites
- High Definition images for large screen devices

## Version 6 Features:

- Option to turn off the device from the screen
- You can choose your own pictures/photos and text for the Housemate cells
- Text to speech now has its own icon in the main menu
- Possible to add/edit/delete contacts
- Possible for the user to choose a photo for a contact
- Text to speech prediction improved
- Compatible with 2.3.3 phones
- Works with LG HBM235 Bluetooth headset
- Number of grids increased to 16, number of codes increased to 250
- Possible to choose the Font size for keyboard keys and SMS messages

#### Version 7 Features:

- Possible to Erase and Test an IR code
- Possible to Erase the flash memory of HouseMate hardware
- Possible to change the Font and image size throughout the application
- Images of contacts and Housemate grids scale to selected image size
- Go-to-sleep option added to main menu for touch screen users
- External switch preference settable directly from the Homepage menu.

#### Version 8 Features:

- Support for multi-part sms messages (messages > 160 characters)
- Possible to send a text from the Dialer in advanced and expert mode
- Typing '\00\' in the dialer at the beginning of a number inserts a '+'
- Possible to quit ClickToPhone from the menu by choosing another input method
- Pressing the power off button disconnects hardware
- Dutch Language now included

#### Version 9 Features:

- Scaling of numbers in Dialer and Telephony keyboards
- Wake-up performance improved on Samsung Galaxy and HTC Flyer tablet
- Android 3.2 ready
- Notification message if bluetooth is not enabled or paired
- Version 9 is the first full release version

#### Version 10 Features:

- Better instructions when recording infrared codes
- Ability to examine the contents of the HouseMate infra-red memory
- HouseMate grids use full screen including notification section
- Ability to change the acceptance time for switch inputs
- Multipart messages (>160) supported

#### Version 11:

- Localised to Italian
- Acceptance time for switch inputs adjustable to avoid accidental presses
- Better performance in stand-alone and tap-screen mode

#### Version 12:

- Two switch scanning
- Two switch advanced scanning (for wheelchair interfaces)
- Five switch joystick mode (ClickToPhone hardware only)
- Ability to pre-select a remote control brand before recording infrared
- Feedback on what infra red codes have been used
- Advanced settings option in HouseMate menu
- You can now create a proper shortcut to ClickToPhone and place it on the home screen

#### Version 13:

- Set-Top mode for using HouseMate as a pure remote control
- User can edit some ClickToGo settings
- Bug fixed which stopped ClickToPhone on phones with a large number of contacts

#### Version 14:

- Call history
- SMS folders: Inbox, Sent, Drafts and Conversations
- Ability to delete an SMS message

#### Version 15:

- Reminder app included.
- Can remind 15min, 30min, 1hour and 2 hour beforehand
- User gets a visual notice and a user defined ring tone
- Switch accessible internet browser
- Mouse pointer
- Zoom controls

#### Version 16:

- Performance improvements
- Version 17 A bug was introduced in version 16 to the Edit Ir function. This has been corrected
- Localised to US English

- Localised to Danish

#### Version 18:

- Improved performance in landscape orientation
- Better scaling of graphics on high density phones
- Ability to select the highlight color for the lists and keyboard keys
- Touch screen preferences added for debounce and repeats

#### Version 19:

- Ability to add bookmarks and enter a URL
- Touch screen preference added for disabling scrolling
- Graphics improved in HouseMate and on Homepage
- SMS Location alert. Sends your location to a pre-selected mobile number.

#### Version 20:

- Improved wake-up performance and disabling of key-guard
- Backup and Restore bugs resolved
- Extra options for SMS alert feature

#### Version 21:

- Auditory scanning of keyboards and word predictions

#### Version 22:

- SMS Location alert performance improved and new option for auto-answer
- Intelligent Scan preference added for better scanning of lists

#### Version 23:

- Possible to use multiple telephone numbers for SMS alert function
- DTMF tones added to Dialer
- Now possible to quickly populate grids in HouseMate with Ir codes
- New default grids: Dvd, TV2 and Keypad added
- Extra sets of icons added to Ir codes

#### Version 24:

- New methods for configuring HouseMate grids
- Batch operations for clearing cells and defining labels etc

- Ability to import Infrared signals of different remote control brands rather than having to record them
- Ability to create you own database of InfraRed signals
- Bug fixed whereby it was not possible to use imported pictures in HouseMate grids
- Ability to specify a backup folder on the SD card allowing different users to use the same device.
- Menu options added to Reminders

#### Version 25:

- Improvments to auditory scanning
- When configuring HouseMate labels are centred in cells when no icons are selected

#### Version 26:

- Download speeds to HouseMate hardware improved
- Talking clock option using text to speech
- Joystick mode improved for full control of mouse pointer using wheelchair joystick or switchjoystick

#### Version 27:

- 'Phone Settings' option now replaces 'Go to sleep' option with the ability to adjust screen brightness, volume, wifi on/off and ringer mode
- Icons added for bed controls

#### Version 28:

- HouseMate options improved
- Wizard added to allow you to quickly create a grid of InfraRed commands
- It is now possible to create grid template and save them to the SD card
- Default templates for TV, DVD, Music, Intercom and House included.

#### Version 29:

- In advanced and expert user levels it is possible to quicky find a contact by scanning an alphabetical keyboard rather than using the page up and page down functions.

#### Version 30:

- Wizard improved for creating new infrared commands
- Comprehensive set of graphics and icons added to HouseMate

#### Version 32:

- First attempt at voice recognition. You can use voice input to write a text.
- Responsiveness of software improved even when trying to connect to hardware.
- Latency issues of switch inputs resolved

#### Version 33:

- Beta software for using voice recognition to control HouseMate

#### Version 34:

- Connection bug introduced in version 32 resolved
- Performance improvements

#### Version 35:

- Localised to Dutch
- Screen orientation problems resolved for Android 3.1 tablet devices
- Support for HouseMate+ (mouse pointer functionality)
- Compatible with Android 4.0

#### Version 36:

- Virtual mouse pointer for controlling ClickToPhone
- Users can now edit ClickToPhone settings by going to PhoneSettings->My Settings
- Status reporting of user activity and critical events by SMS
- Diagnostics file
- Auto reconnect to hardware even if bluetooth disabled and unpaired.
- 

#### Version 37:

- Keyboard layout improved on high density displays (such as Galaxt Note)
- Confirmation required to send an SMS alert
- GPS location can be included in status reports
- Bug in status reporting resolved (with no phone number specified a crash would occur)

#### Version 38:

- Improvements to Expert mode and Mouse pointer mode

- Email added. You can now email a contact from the contacts page in expert mode.

#### Version 39:

- Compatible with Android 4.0.3
- Improvements to Joystick mode
- Improvements to rendering of imported images in HouseMate on high density displays

#### Version 40:

- You can now import a template grid and download the signals to the HouseMate hardware in one operation.
- Improvements to SMS messaging
- All phone contacts are now shown in ClickToPhone.

#### Version 41:

- Backup & Restore reworked
- Ability for expert user to restore backups independently (via Phone Settings option)
- Control of HouseMate+ mouse pointer using a crosshairs
- Ability to "swipe" screen in different directions (HouseMate+ only)
- Preferences re-organised
- Reminders bug fixed
- HouseMate backup speed & reliability improved
- User cancelable low battery warning at 20%
- Automatic pairing and connection once hardware requests PIN (for easier setup)

#### Version 42:

- Support for 5 switch version of HouseMate+ for wheelchair interfaces

#### Version 43:

- Beta support for Z-Wave. Category 3 devices only at present.

#### Version 44:

- Support for controlling Z-Wave Scenes

#### Version 45:

- Support for Z-Wave category 2 devices (dimmers).
- Improved wake-up performance
- Improved bluetooth connection and pairing on ICS devices.

#### Version 46:

- Setup Wizard introduced
- Ability to completely customise the clicktophone homepage
- Preferences re-organised to accomodate changes

#### Version 47:

- Ability for HouseMate+ users to launch and control Notification window
- HouseMate preferences relocated back to main Preferences page (sorry!)

#### Version 48:

- Partial translation into French and German
- French and German dictionaries
- ClickToPhone can now be controlled by a Point-It! bluetooth joystick

#### Version 49:

- Customisation features (wall-paper, key background etc, disable status bar)
- Partial translation into French with Azerty keyboard
- Partial translation into German
- Manuals included with installation and saved under  
sdcard/clicktophone/extras/docs
- ClickToPhone apk bundle included with installation and saved under  
sdcard/clicktophone/extras/bundle
- Performance improvements

#### Version 50:

- Stability improvements for Android 4.1.1 (Jelly bean)
- User manual now accessible from within application
- ClickToPhone apk bundle automatically installed during installation
- Setup wizard launched after installation.

#### Version 51:

- Further improvements for Android 4.1.1 (virtual mouse pointer operational)



#### Version 52:

- ClickToPhone now accessible from Application list
- Input method checked on startup

#### Version 53:

- You can add InfraRed shortcuts to the ClickToPhone homepage
- Application automatically restarted on certain config changes (key colors)
- Full French text to speech scanning

#### Version 55:

- Full scanning music player and picture gallery added.
- Possum Primo codes added.

#### Version 56:

- Improved installation wizard and step by step guide.
- ClickToPhone can now be opened after installation and launched from the application list.
- Manual can be opened from the application list.
- KNX and Beam-It! codes added (requires hardware serial number > 165)
- SMS messages can be sent to more than one recipient.
- Auto responder for SMS alert function.
- Uninstall instructions

#### Version 57:

- SMS messages can be forwarded to another contact
- Cursor controls for Expert users when writing SMS
- Auditory scanning improved (now speaks out row numbers)

#### Version 58:

- Support for Curtis wheelchair joystick controls

#### Version 59:

- Improvements to alerts
- Support for Android 4.1.2 devices
- Support for 2.1 devices dropped
- Google hardware filtering over-riden
- Bug in favorites corrected

- Zoom button added to gallery
- Autoplay of next track in music player
- Automatic configuration of preferences when connecting to a Curtis joystick.

#### Version 60:

- Improvements to joystick mode
- Better handling of telephony keypad
- More robust handling of licencing
- Alert triggers the alarm ir code if connected to HouseMate (ir code 0)
- Pointer mode launch improvements (will launch from long click within ClickToPhone only in technician mode).

#### Version 61:

- Bug fix in contact adder form
- Advanced bluetooth setup options

#### Version 64:

- Batch recording of InfraRed signals
- Improved handling when a backup of InfraRed signals fails.
- Configuration of back keys and exit keys in HouseMate
- More options for handling and configuring alerts including read confirmation
- Bug fixes in SMS receiver when app not running
- Bug fix in Call History

#### Version 66:

- Translated into Finish
- Screen re-orientation issues resolved when recording signals
- A bug was introduced into the InfraRed backup/restore process in version 64.  
Sorry. This is now fixed.

#### Version 67:

- Wheelchair interfacing options for joystick only operation.
- Headset switch can be used for scanning for demonstration purposes.

#### Version 68:

- Minor performance improvements

#### Version 69:

- Possible to launch mouse mode from homepage
- Joystick can be used with arrow keys for easier navigation of lists and home screen
- Improvements to text to speech read out of SMS messages
- HID preferences re-organised into separate category

#### Version 70:

- Minor performance improvements

#### Version 71:

- Ability to disable the phone's home key
- More customisation options (font, background, keyboard key colors)
- 2 new themes added in wizard
- Integrated with Opera Mini web browser. If installed Opera Mini launched by default.
- Ability to perform continuous scan of up/down/left and right keys.

#### Version 72:

- Cross hairs customization options
- Mouse click lock
- Options for controlling third party AAC apps (such as Predictable, AlexiCom etc.)

#### Version 73

- Missed call notification
- Starred contact list
- Voice recognition for speech to text input into other applications
- More options for controlling third party apps
- Preferences reorganised (sorry!)

#### Version 74:

- Text to speech improved
- Categorised phrases (quick texts) with symbols
- Ability to insert a phrase into an SMS message
- On screen soft keys for home/menu/back for joystick users
- Mouse mode simplified for joystick users

#### Version 75:

- Minor performance improvements
- Support of Android TV dongles
- Joystick flick mode

#### Version 76:

- The home page can be displayed as a grid
- The home page contents can be re-ordered
- Applications can be added to the homepage in expert mode.

#### Version 77

- More customisation options and color settings
- Dialer layout improved
- Joystick mode improved
- Control of other apps in Tap-the-screen and Headset-switch scanning methods.

#### Version 78:

- Updates for Android 4.3
- Improved handling of hardware settings conflict
- Improvements to joystick flick mode (hardware will be kept awake during use)
- Bug fixes

#### Version 79:

- Soft keyboard icons re-vamped
- In expert mode, you can add the sms you write to the phrase book
- Z-wave disabled in some jurisdictions without full licence

#### Version 80:

- Technician mode option moved to main Project Settings page
- Original preferences listing re-instated under Project Settings for backwards compatibility. This will be removed in a future version.

#### Version 81:

- Support for AbleNet's Blue 2 switch interface
- Graphics improved for tablets and tv dongles
- Headset switch scanning mode improved

Version 82:

- Schneider and Bang&Olufsen infrared codes added (HouseMatePro only)
- Ability to reply to an SMS added within conversations page
- Graphical errors fixed
- Version to be released on the Play Store

Version 83:

Version 84:

- Problem playing YouTube within Internet app resolved

Version 85:

- EasyWave added to HouseMate for control of Eldat devices
- Problem solved in mouse pointer mode (ANR if click lock enabled in swipe mode)

Version 86:

- Bug introduced in version 85 resolved. A crash would occur in Contacts/Messages or Call History if a contact's phone number contained a parentheses - SORRY!

Version 87:

- Keyboard icons can be displayed in greyscale
- Further translation into Dutch
- A bug was introduced from Version 79 that would intermittently cause a crash on high density displays (S3 & S4). This has been corrected.

Version 88:

- Updates for Android 4.4 (KitKat)

Version 89:

- Big keyboard option in text to speech
- Wizard steps introduced to pair the HID module
- Option to disable preview of keyboard keys
- Infrared codes names shown when transmitting and making backups
- User manual updated
- Bug corrected in Alert function

#### Version 90:

- Full translation to Finish
- Updates for Android 4.4.2
- Grid layout is now default style
- Bug corrected in HouseMate which caused temporary disappearance of grid when editing labels

#### Version 91:

- Compatible with new HouseMatePro Mk2 hardware
- HouseMatePro Mk2 supports bluetooth auto-reconnect which saves battery life and improves performance
- Security features strengthened (more persistent nag messages)
- Wizard improvements for pairing HouseMate and HouseMate HID

#### Version 92,93,94:

- Updates for connection to Curtis instruments handsets and multifunction displays
- Partial translation into Norwegian

#### Version 95:

- Memory leak bug fixed in status reporting which affected performance on KitKat devices.

#### Version 96:

- Ability to clear the bluetooth pairings in HouseMatePro Mk2 hardware making it easier to repair with other devices.

#### Version 97:

- Improvements in reliability and speed of making backups of InfraRed codes.

#### Version 99:

- Full KNX AP421 database of IR codes added
- Bug fixes

#### Version 100:

- Bug fix affecting Samsung Tab 3 users

Version 103:

- Bug fix affecting Samsung Tab Pro 8.4"
- Translation into German and Swedish
- Improvements to bluetooth connectivity and pairing wizard
- 60 day trial period before nag messages appear

Version 104:

- Bug fix when adding new contact from call history or messages

Version 106:

- Improvements to bluetooth connection reliability
- Updates for Lollipop
- Bug fix when deleting phrases

Version 107:

- Integration with dwell click app

Version 108:

- Further enhancements for Dwell Click app

Version 109:

- Ability to add more housemate grids
- Ability to transmit an IR code as you answer a call
- Ability to see which EasyWave and/or Z-wave functions have been used and by which commands
- Feedback from Z-wave
- Phone settings page can be displayed as a grid and re-ordered
- More scanning options: -Scan single rows twice -Pause on first item -Go back one step with a second switch
- Starred contacts added to all user levels
- You can have Go to sleep option on homepage as well as phone settings

Version 110:

- New scanning methods introduced for mouse pointer mode
- Ability to Edit the dictionary and import new words
- Ability to open the Vera (Z-Wave) Dashboard from within ClickToPhone to add devices and create scenes
- Ability to automatically find Vera on your home network
- Compatible with Vera Edge

#### Version 111:

- Phrase book can be added to homepage for quick text-to-speech
- Improvements to joystick mode when controlling the mouse pointer
- Easywave and Zwave commands can be cleared
- Housemate advanced settings reorganised
- Full translation into French

#### Version 112:

- Color picker added and appearance preferences reorganised
- Telephony splash screen added
- Simple status bar added
- \"Fast typing\" option added to scanning
- Two character search of Contacts
- Screen can sleep in \"Never when connected\" hardware mode
- Icons added for sms sent/received/read/draft.

#### Version 113:

- Color palette added
- Cardinal mouse movements re-instated

#### Version 114:

- Hardware settings displays firmware version and whether or not EasyWave module is fitted
- Improvements to EasyWave programming
- Position of mouse pointer icon can be selected in Pointer preferences

#### Version 115:

- Improvements to bluetooth connectivity
- Supports Mk3 hardware
- Updates for os 5.1

#### Version 116:



- Improvements to bluetooth connectivity
- Audible confirmation of connection to HouseMate HID
- Error handling when connecting to multiple devices

Version 117:

- Further improvements to bluetooth connectivity
- Bluetooth error handling on pre-KitKat devices

Version 118:

- Bug fix in phone settings when using joystick↵ input

Version 119:

- Improvements to telephony handling and telephony splash screen behaviour

Version 120:

- Bug fixed when answering call - was displaying a call failure message in some instances.

Version 121:

- Further improvements to Mk3 bluetooth connectivity. Will attempt 3 connections after hid has connected.

Version 122,123:

- Further improvements to bluetooth connectivity and error handling.

Version 124:

- From version 118 a bug was introduced whereby if, after installation, an attempt was made to connect to hardware before any pairings were made, the app would stop. This has been resolved in Version 124.

Version 125:

- Back key acts as menu key in homepage (for devices with no menu key)
- In previous versions the SpecifySppUUID option could not be used with \"wait for

hardware\" connection method. This has been resolved. Required for most Motorola devices and CAT B15Q tough phone.

Version 126:

- Quicker connectivity when answering a phone call.
- Connection method can be chosen from the homepage menu key, rather than through preferences.
- Hardware Camera key can be disabled under Behaviour preferences->Hardware Buttons.

Version 127:

- Bug in whether virtual soft keys appear or not has been resolved.

Version 128:

- Improvements to Z-wave functionality.
- Improvements to Internet browser.
- New InfraRed code databases.
- New batch operations in HouseMate to erase multiple ir signals and clean up your project.
- HouseMate cells can be programmed to play an audio clip when selected, like a communication grid.
- Voice input available in Text to speech app and when controlling third party apps.
- Additional scanning speeds.
- Improvements to alert functionality. Low battery can now trigger sms alerts.

Version 129:

- New features to support HouseMatePro Mk4 devices
- Http commands can be entered into help text of housemate cells
- Quick Dial option added in Telephony preferences for beginners
- 5 minutes pre-alert option added to Reminders.

Version 130:

- New features to support Z-Wave direct
- Bug fixes and improvements
- Partial translation to Portuguese

Version 131:

- Improvements to joystick mode during calls

- Added \"Expert\" to Telephony Keyboard options. It is now possible to go into mouse mode during a call that you made OR answered.
- Support for ClickToPhone Version 3 hardware

#### Version 132:

- Support for BeNext Z-Wave gateway (beta version)

#### Version 133:

- Improvements to Internet & Bookmarks
- Quicker mouse click in joystick mode
- Telephony option added to Wizard
- Black & White is default theme
- Auto-answer option in Phone Settings

#### Version 134:

- Z-Wave setup wizard added
- Links to TeamViewer Quick Support added to project menu
- Frequency of use keyboard added
- Tawasol Arabic symbols added courtesy of [www.tawasolsymbols.org](http://www.tawasolsymbols.org)
- Speech input added to Internet app
- New preferences: HouseMate preferences->Jump to back key
- Scanning Preferences->Row/Column & Acceptance Delay
- Pointer preferences->Behaviour->Icon Preview

#### Version 135:

- Support for Z-Wave groups added
- Support for MouseMate (bluetooth PC link)
- New theme added
- Time out for menu button and better handling of back key
- New preferences: Homepage preferences->Mouse control
- Pointer preferences->Back key
- Telephony preferences->Fixed line IR phone

#### Version 136:

- Wakeup command option added to Z-Wave direct settings

#### Version 137:

- Improvements to Z-Wave group functionality (toggle and push-button options)

- Network scanning improved and faster
- Easywave alarm bug handled (firmware 3-6)
- Wheelchair toggle option added to all user levels except beginner
- Pairing wizard improved (automatic detection of back key method)
- Fixed line telephone keypad added (open the example project in HouseMate)

#### Version 140

- App Optimisation part of setup wizard

#### Version 141-144

- Telephony keyboard macro option for controlling native dialer app.
- Easywave firmware patch option
- Drag lock option
- Disable App Optimisation option
- Automatic HID pairing
- Option added to allow user to Pair/Unpair HID
- HID configuration options added (Profile, PIN)
- Improve HID Discoverability
- Hardware Alarm triggers SMS Alert

#### Version 145

- Pairing Wizard introduced. App automatically pairs with both HouseMatePro and HouseMate HID

#### Version 146

- Support for Kindle Fire
- Improved configuration of Z-Wave groups
- Mouse mode disabled when scanning method set to Direct Access

#### Version 147

- Contacts grouped when name and number are the same

#### Version 148

- SMS messages shown as list of sent/received text “bubbles” when you open a conversation thread.

#### Version 149

- Support for IFTTT events ([www.ifttt.com](http://www.ifttt.com))
- VisoCall IP Infrared codes added to database
- API introduced for 3<sup>rd</sup> party apps that wish to trigger HouseMate signals.

#### Version 150

- Bug fixed in new SMS history view

#### Version 154

- Updates for Android 6.0
- Support for Eldat wireless switch control
- Full translation into Dutch

Version 157

- New Swipe method added

Version 158

- Mouse Mode added to Telephony Behaviour options
- Mouse Speed option added to Phone Settings
- Voice input added to Gmail
- Updates for Version 15 hardware

Version 159

- Bug fixed in Applications
- 'Telephony Preferences->Answering Calls' replaces 'Headset hook' option.

Version 160 (10/1/17)

- Additional methods for answering calls

Version 161 (11/1/17)

- Support for Hands Free Protocol (requires hardware with firmware > 16)
- Enhancements to Telephony preferences->Answering Calls

Version 162 (31/1/17)

- Support for ClickToPhoneV4 hardware

Version 164 (21/2/17)

- End call options added to Telephony preferences

Version 165 (21/2/17)

- Option added to remove screen overlays when clicking

Version 166 (7/3/17)

- Setup wizard automatically launched after install wizard completes
- Updated HouseMate+ units can utilise "Wait for hardware" connection method.

Version 167 (16/3/17)

- New "software only" method added to answer phone calls (HookControl).

Version 169 (30/3/17)

- Z-Wave Direct Wizard added to support functions of new z-wave module
- Improvements to HookControl app.

Version 170 (3/4/17)

- Improvement to HookControl installation guide
- Off-line configuration of z-wave direct

Version 171 (19/4/17)

- Improvements to Pairing Wizard
- Updates for HouseMatePro Mk5 for full DTMF support
- Integration with GigaSet HC450 door opening system
- Timeout to switch control when using Voice input in HouseMate
- Azerty keyboard option added

Version 172-176

- Performance improvements

Version 177 (20/9/17)

- Integration with Android 7 accessibility features
- MouseMate can now be used with HouseMate HID Mouse/Combi
- External Device option added for control over a second device (iPAD or Android) using HID Combi or Keyboard.

Version 178

- Support for iSwitch and other Bluetooth keyboards
- Z-Wave over Wifi disabled for new projects

#### Version 181

- Ability to audition an IR code in a signal database

#### Version 182

- Support for HouseMate S devices
- Smart Connect mode
- DTMF enable/disable option for S devices
- Improvements to Selection method UI
- Grid option for mouse control.

#### Version 188 (20/3/18)

- Book reader added
- Keyboard option added to MouseMate
- Sequential clearing of buttons in a grid
- Z-Wave direct dimming options added
- Try again button added in pairing wizard
- Appearance of HouseMate grids improved
- Bundled apps removed
- Manual replaced by link to website
- IFTTT disabled for new projects
- New keyboard key colors
- Licence no longer required for special features

#### Version 189-192

- Grid type interface added for controlling the mouse pointer and cross-hairs in expert mode.

#### Version 193

- Option to enlarge the size of the mouse grid
- Apps started from Applications launched with mouse mode active rather than the soft-keyboard.

#### Version 194-198

- Updates for Android 8
- Improvements to Accessibility Support Package
- Ability for cross-hairs to appear on top of notification slide and other system dialogs in Android 8.
- Beta Voice Commands app bundled in package

#### Version 199

- Voice Commands unbundled from package and provided as a separate app on the play store.

#### Version 200

- Updates for firmware 20+ hardware

#### Version 201

- Dutch KPN signal database included

Version 202 (24/8/18)

- Sky+HD IR brand introduced and ability to set the "detector mode" in hardware running firmware 21 or higher.
- Dutch KPN numbers signal database included.